D

Securafil LC®

Gebrauchsinformation

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Securafil LC®

Instruction for use

Securafil LC is a light-curing two-component glass ionomer filling material.

MIXING

Shake powder before proportion. One measuring spoon is used for one whole drop of liquid (hold bottle upright and squeeze out one drop without air-bubbles). Mix first with one half of the powder and go one while taking half of the remaining powder and so on. Work powder portion by portion into the liquid till the desired consistency is arrived.

APPLICATION

Prepare the cavity, clean it thoroughly and dry well. Place the required amount of material onto a mixing pad, shortly before applying it. Avoid exposure to daylight or to the operating light as this can cause the material to harden prematurely. Apply the material in layers of 1mm and polymerize these also layer by layer.

The material is to be cured by using halogen-light. The light guide should be positioned as close as possible to the filling material. When the light guide is 2cm above the surface, cure the material for at least 40 seconds. Possible tenacious remains of the material on the enamel should be removed by using a rotary instrument.

SIDE EFFECTS

Undesirable side-effects to this medical-product are not to expect, if working corresponding to the instructions of use. In case of side-effects -even when doubtful- please inform us with exact description of the circumstances and symptoms.

INSTRUCTION REFERENCES

Take care of all working- and caution advices. In case of unexpected side effects or other reactions (even if work was made according to the Instruction of use) please inform us, if possible, with exact description of the accompanying circumstances and symptoms.

CONTRAINDICATIONS

Minimal toxicological risk. Some patients may be sensitive to components.

ACTIVE INGREDIENTS

Liquid: polyacrylic acid, water, methacrylate.
Powder: Calcium-aluminium-fluoro-silicate, barium glass

STORAGE

Store at room temperature up to 24°C.

SHELF Life

When stored properly until expire date (2 years from date of production, 6 months after opening).

PRESENTATION

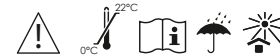
REF WP4144 1 x 15g powder, 1 x 8g liquid + accessories
REF WP4124 1 x 7g powder – 1 x 5g liquid + accessories

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